

MiniMed™ 780G System at a glance





This quick guide covers the main functions of the MiniMed™ 780G system when the automatic SmartGuard™ function is active. For more details on how to use the product, refer to the system's instruction manual.


Shortcuts




To stop the entire insulin delivery or bolus, press **UP**. You can continue delivery from the same location.

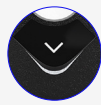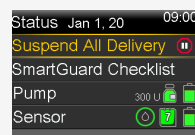

You will find the bolus screen by pressing **DOWN**.

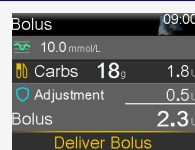


You can access the 'Time in Range' by pressing **RIGHT**.

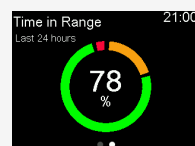


Select Button



Notification Light

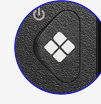

You can view the sensor graph by pressing the **DIAMOND BUTTON**.

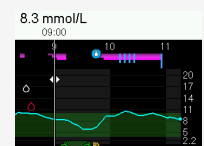

Menu

- Press the **SELECT** button on the Home screen
- The Insulin icon will show as selected
- Move to the desired icon: it changes colour and the menu name appears at the top of the screen

HISTORY & GRAPH History Sensor Glucose Review Graph Time in Range	SMARTGUARD SmartGuard Checklist Temp Target SmartGuard Settings SmartGuard On and	SOUND & VIBRATION Silence Sensor Alerts Volume Sound Vibration Alert Settings shortcut
RESERVOIR & SET New Reservoir & Set New Reservoir Only New Set Only Fill Cannula	INSULIN Bolus Basal Suspend All Delivery Delivery Settings shortcut	BLOOD GLUCOSE BG
STATUS Suspend All Delivery SmartGuard Checklist Pump Sensor	PAIRED DEVICES Pair New Device Pair CareLink	SETTINGS Alert Settings Delivery Settings Device Settings

Checking your glucose levels

The Guardian™ 4 sensor does not need to be calibrated but there may be occasions when you may need to check your glucose levels with a BG meter. When this occurs, the pump will display an **Enter BG now** alert.

Press the **back button** to send the BG value to the pump without delay.




You can also enter blood glucose to the pump manually:

1. Choose  from the menu.
2. Enter BG
3. Select SAVE.

Temporary target

When needed, you can set a temporary target (8.3 mmol/l) for the pump from the **SmartGuard™ menu**. When the temporary target is active, auto correction boluses are not delivered.

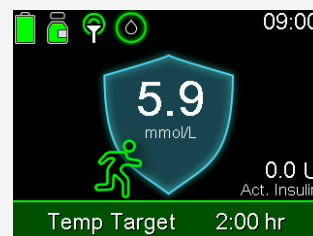

Pump home screens


Blue shield:

SmartGuard™ function is active and works normally.



Red bar:

Insulin delivery is suspended.

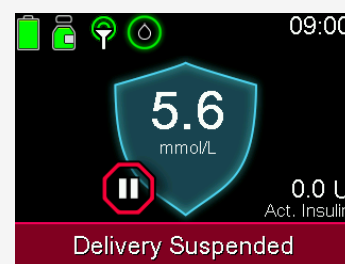


Note: If the pump is in manual mode, the shield is not shown on the home screen. The pump also works a little differently. Confirm the instructions for manual mode separately.

Pump alerts and all alarms

All alerts, alarms and messages given by the pump must be acknowledged without delay.


If necessary, follow the instructions displayed on the pump screen.

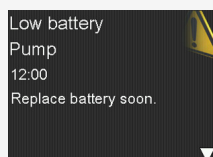

Alert:

Your pump needs attention.

E.g. Low battery. Low reservoir.

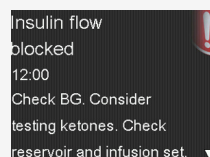

Alarm:

Address right away.

E.g. Insulin flow blocked. Replace battery now.


Note: The notification light flashes when the pump has an alarm or alert.

Staying in the SmartGuard™ feature:

When the pump requires an action to stay in the SmartGuard™ feature, it provides a notification and the time remaining before an exit to the manual mode (max. 4 hours).

- 1) Read the notification from the bar.
- 2) Respond to the action needed.

If the pump exits to manual mode, confirm the instructions separately.

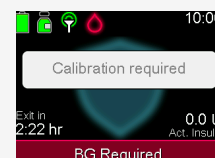

Medtronic

Medtronic Australasia Pty Ltd
2 Alma Road
Macquarie Park, NSW 2113, Australia

Tel: +61 2 9857 9000
Fax: +61 2 9889 5167
Toll Free: 1800 777 808

E-Mail: australia.diabetes@medtronic.com
Facebook: www.facebook.com/MedtronicDiabetesAUS
YouTube: [Medtronic Diabetes ANZ](https://www.youtube.com/MedtronicDiabetesANZ)

<https://www.medtronic-diabetes.com.au/>

Always read the instructions for use.

For detailed information regarding the instructions for use, indications, contraindications, warnings, precautions, and potential adverse events, please consult the device manual. For further information, contact your local Medtronic representative.
© 2021 Medtronic Australasia Pty Ltd. All Rights Reserved. Please note: In contacting the Diabetes Toll Free, your personal and health information may be disclosed to an operator located outside Australia. 10250-102021